



Sterilization Instructions

Sterility:

Restocking:

Crowns exposed to blood and saliva should be cleaned of debris and steam autoclaved prior to restocking.

To remove debris, contaminated crowns may be immersed in 3% hydrogen peroxide for one minute then rinsed thoroughly with air/water spray.

Place crowns in a sterilization pouch. Steam autoclave at 121C(250F) for 30min, or 135C(275F) for 10min.

916.677.1447

6140 Horseshoe Bar Road, Suite L
Loomis, CA 95650 USA

sprigusa.com